



1. Product name

Southpaw Cocoon swing

2. Product code

18720065

3. Colour

As shown



4. Short description

The Southpaw Cocoon Swing is an innovative sensory swing from Southpaw, specially designed for children who need sensory stimulation, relaxation, and a sense of security. Thanks to its smart design, this swing offers both fun and therapeutic benefits, making it ideal for use in schools, therapy rooms, sensory spaces, and hospitals.

5. Contents

- 1x Southpaw Cocoon swing
 - 1x stabilizer board
 - 1x safety snap
 - 1x instruction sheet
 - 1x safety snap instruction sheet
 - 1x maintenance checklist
- WORKING LOAD: 100 lbs. per layer / 400 lbs. total.

6. How to use

WORKING LOAD: 90 kg / 200 lbs. total.

Hanging Instructions:

- Attach the safety snap to the loops on top of the Cocoon Swing and then secure the safety snap to your suspension point.
- The **Southpaw Height Adjuster (13626652)** allows you to easily adjust the height of the equipment to suit your needs.
- The Cocoon Swing can be filled with different types of material, such as balls or other suitable fillings.

Warning:

Failure to properly install a forged steel eye bolt (1/2" or 5/8") may result in serious injury or death. If you have any questions regarding the installation of the forged steel eye bolt, please contact us.

7. Maintenance

Monthly checks (record on checklist):

All the following wear points should be checked on a monthly basis and recorded on the checklist:

- Check for ripped seams or loose stitches. If found, discontinue use until repaired.
- Check wooden parts for cracks and splinters. If cracks are present, discontinue use until repaired. Sand splinters smooth.





Matting – Precautions:

Always use appropriate crash mats under and around suspended equipment.

Storage

- Store away from direct sunlight
- Keep away from chemicals and cleaning agents
- Not accessible to clients without supervision

Cleaning

- Check with your infection control department before using these procedures.
- Check first for client allergies.
- After all wooden parts have been removed, the fabric may be machine washed in cold water with mild detergent. See the back side for instructions on removing the wood from the fabric.
- Tumble dry on low heat.

TO REMOVE THE COVER:

- Open the door of the swing and turn the top portion inside out.
- Locate the opening in the purple fabric located at the junction of the purple and the black fabric.
- Remove the screw and washers holding the two ends of the circular piece of wood together with a 7/32" hex key wrench.
- Slide the circular piece of wood out of the fabric.

TO REPLACE THE COVER:

- Open the door of the swing and turn the top portion inside out.
- Locate the opening in the purple fabric located at the junction of the purple and the black fabric.
- Slide one end of the circular wood piece into the opening in the purple fabric located at the junction of the purple and the black fabric.
- NOTE: one end of the wood circle has a threaded insert in it. The insert has a "washer" on one side (**fig. 1**). The screw goes into the opposite side from the washer.
- When putting the wood circle into the fabric, be sure that the washer is facing the black fabric. Slide the fabric around the wood circle until both ends are at the opening (**fig. 2**).
- Put the split lock washer, then the flat washer on the screw. Put the screw into the hole without the threaded insert.
- Overlap the ends of the wood until the holes are aligned and start the screw into the threaded insert and tighten securely (**fig. 3**).
- Turn the fabric right side out and hang.

Figure 1

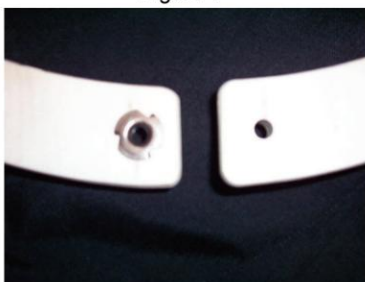


Figure 2

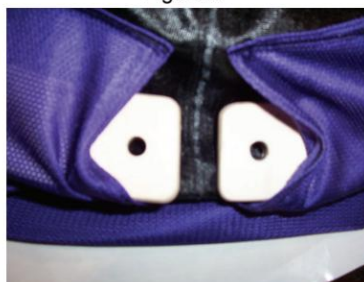


Figure 3





8. Safety

The safety of your clients depends on carefully selecting activities that match — and challenge — their physical abilities.

Always use proper spotting and safety techniques. In addition, the layout of the room, correct installation, and regular maintenance of your equipment are critical.

Southpaw equipment is not intended for recreational or outdoor use. Under no circumstances should this equipment be used without supervision.

This document is for example purposes only and in no way replaces local, regional, national, or international safety standards. Please note: it is your responsibility to obtain all relevant safety information that applies to your facility.

For more information, you can visit our website www.nenko.com

