

## 1. Name of the product

Multi Memo Recording Disc

## 2. Product code

20361471

## 3. Colour

See image



## 4. Brief description

This recording disc can record and play back multiple messages via the built-in microphone and speaker, with a total recording capacity of 6 minutes (360 seconds). Up to 60 voice messages can be recorded, saved and then played back one by one using the forward and backward buttons.

## 5. In use

### Activating Multi Memo Recording Disc

1. On the back of the Disc is an on/off button.

### Recording message

1. Press the record button (red button) to record a message.
2. When the red light turns on, recording has started. You will also hear a "beep" sound once to confirm that the recording has started.
3. Speak the desired recording clearly near the microphone, with a distance of about 15 centimetres.
4. Finished recording? Do not release the record button until the desired message has been recorded. You will hear two beeps as confirmation.
5. It is possible to record for 360 seconds with this Recorder. In addition, up to 60 separate messages can be recorded.

### Play recorded message

1. Press the play button (black button) to play the last recorded message.
2. Press the play button again to stop the message.

### Finding recorded messages

1. Press the "next" or "previous" button to find the desired recording. Repeat this until you have found the desired message.
2. Do you want to go directly to the last recorded message? Press the "next" button for three seconds.
3. Do you want to go directly to the first recorded message? Press the "previous" button for three seconds.

### Deleting a recorded message

1. Find the desired recording to delete.
2. Press the "delete" button (blue button) once to delete this message.
3. The red light will light up and you will hear a beep once. This is the confirmation that the deletion was successful.





***Please note:***

- Do you want to delete all messages? Then press and hold the "delete" and "next" buttons simultaneously for 15 seconds. The red light will light up and you will hear a "beep" sound once, this is the confirmation that you succeeded in deleting all recordings.
- Do you want to keep the recorded messages? Slide the "lock" slider on the back of the Multi Memo Recorder.

## **6. Sense Stimulations**

Feel, Hear

## **7. Specifications**

Size: 8 Ø cm

Battery: 3 x LR44 (included)

## **8. Included**

1 Multi Memo Recording Disc

3 x LR44 Battery

## **9. Important information**

This item is not suitable for children under 3 years of age.

\*Danger of suffocation

Use the correct batteries (LR44) to prevent damage.

Do not let water get into this product.