



1. Name of the product

Talking Tin – Yellow 20 seconds recording

2. Product code

20363471

3. Colour

Yellow



4. Brief description

Records and plays messages, music or sound effects. A challenging way to stimulate talking and listening. Fast way to make something verbally clear. Compact, fun and practical original designed to help visually impaired people identify canned food. Magnetic. 20 seconds recording time.

5. In use

Activate Talking Disk

1. To activate the battery, pull out the white strip on the back of the disk

Recording message

1. Hold the disk with both hands and press both thumbs on the two red buttons at the front, press these buttons.
2. When the red light turns on, recording has started. You will also hear a "beep" sound once to confirm that the recording has started.
3. Speak the desired recording clearly near the microphone, with a distance of about 15 centimetres.
4. Finished with recording your message? Release the two red buttons. You will hear two beeps to confirm the message is recorded.

Please note:

- Make sure the red light stays on while recording to make sure the Talking Disk is recording.
- You can record a new message as many times as you like. As soon as a new message is recorded, the previous recording disappears.
- Do you want to save recorded message? Slide the lock slider on the back to the red Talking Disk.

Play recorded message

1. Press the play button (black button).
2. The recorded message can have a maximum duration of 20 seconds.

6. Sense Stimulations

Feel, Hear

7. Specifications

Size: 7,9 x 2,2 cm

Battery: 3 x LR44 (included)





8. Included

1 Talking Tin
3 x LR44 Battery

9. Important information

This item is not suitable for children under 3 years of age.

*Danger of suffocation

Use the correct batteries (LR44) to prevent damage.

Do not let water get into this product.

For more information you can visit our website: www.nenko.com.

