



1. Name of the product

Sensory Wagon with LED screen

2. Product code

21967010

3. Colour

See image

4. Brief description

For safety and service reasons, this Snoezelen Wagon with LED screen is delivered in a wooden transport case.



*We recommend keeping the wooden transport box for possible service purposes.

1. Open the transport box with a Phillips screwdriver.
2. Release the brakes on the front wheels at the bottom of the Snoezelen Wagon and drive the car out of the transport box.
3. Hang the fibre optics next to the cart.
4. You can see the key that opens the front and back of the Snoezelen Wagon. The key of the Snoezelen Wagon is in the handle of the low door, it is attached with a tie wrap.
5. Open the low door at the front of the Snoezelen wagon with the key provided.
6. Take the liquid projector and the laser star projector from the cabinet and place them on the cart. The liquid projector can be found in the pull-out area of the Snoezelen Wagon.
7. Connect the projectors at the top of the cart to the relevant products, the small cable to the grey laser star projector and the thicker cable to the liquid projector.
8. Take the 3-metre power cable out of the cabinet and insert one end into the socket on the outside of the Snoezelen Wagon (right-hand side as seen from the front) and the other end into the power point in the room where the Snoezelen Wagon will be used.
9. Then take the Adaptor (connection for the bubble tube) out of the cabinet and place it in the other connection which can be found on the outside of the Snoezelen Wagon under the fibre optics.
10. Open the backside of the Snoezelen Wagon with the enclosed key. Here you will find a removable part with the bubble tube and a mirror at the back. This wooden "panel" can be easily removed by gently pulling it backwards by the handle.
11. Place the wheeled section with the bubble tube next to the LED screen and lock the wheels of the wheeled section before use.
12. Fill the bubble tube for 80% of water, leaving a bit of room for the bubbles.
13. Plug the other side of the small power cable, which has just been placed on the outside of the Snoezelen Wagon, into the bubble tube so that it too is supplied with power.
14. Open the cabinet at the front of the Snoezelen Wagon again; in this low cabinet you will find a switch box with 6 red buttons. These buttons are the on/off buttons for the electronic products and components that are used in and on the Snoezelen Wagon. The diagram below shows which button is for which device (from left to right).

Please pay attention: Always plug the Snoezelen Wagon on the outside first and only then switch on all the red buttons!

Button 1 external connection for bubble tube





Button 2 Fibric optics

Button 3 LED screen

Button 4 liquid space projector and Laser Stars Projector

Button 5 DVD-player

Button 6 Aroma diffuser

15. The LED-screen can of course only be used at the same time as the DVD player. The other materials can be used separately.
16. Don't forget the cardboard box provided! You will find it in the lower box. In this box you will find the remote control for the DVD player, a DVD, 2 effect wheels and 4 scent mixtures (these scents can be used with the aroma diffuser).
17. Place an effect wheel in a LED projector and check that all the buttons on the projectors are set to "on"
18. Put 4 drops on one of the fragrance mixtures on the aroma pad at the bottom of the aromatherapy spreader. Unscrew the bottom of the aroma diffuser, put 4 drops of a fragrance mixture of your choice on the aroma pad and close the bottom of the aroma diffuser.
19. The Snoezelen Wagon is now ready to use!

5. Sensory Stimulations

Feel, Smell, Hear and See

6. Specifications

Size: 65 x 60 x 147 cm

Power supplies: 220 V

7. Included

- 2 x Projector
- 1 x Bubble tube
- 1 x Fibre optics
- 1 x Light Source for fibre optics
- 4 x Essential oil
- 2 x Effect Wheel
- 1 x DVD
- 1 x Aroma diffuser