



1. Productname

Nenko Sensory Wagon with LED screen

2. Product code

21970010

3. Colour

As shown

4. Short description

This Snoezel sensory wagon with LED screen is delivered on a pallet for safety and service reasons.

5. Sensory stimulation

Touch, smell, hear and see

6. Use

1. You will receive the Snoezel wagon on a pallet.
2. Carefully move the wagon off the pallet.
3. Remove the key and open the door. The key is located on top of the wagon.
4. Fill the bubble tube with water. Make sure not to overfill the tube and prevent water from spilling outside the tube to avoid damage to the electronic components. Please refer to the additional instructions for bubble tubes on our website.
Attention: Do not connect the Snoezel wagon to the power supply until the tube has been filled with water.
5. Remove all products from their packaging and connect the loose plugs to the corresponding products.
6. The plug of the section with the bubble unit can be connected to the side of the Snoezel wagon.
7. Inside the cabinet there is a control panel that allows the products to be switched on or off.
Attention: The products themselves may also have a separate on/off switch.
For the control diagram, please refer to the supplied diagram.
8. Connect the wagon to the power supply. It is now ready for use.

7. Specification

Dimensions: 60 x 75 x 145 cm

Power supply: 220 V

For more information, you can visit our website www.nenko.com

