



## 1. Product name

Southpaw sling seat

## 2. Product code

29154065

## 3. Colour

As shown



## 4. Short description

The Southpaw Sling Seat has been specially developed to provide comfort and support during therapeutic activities. While traditional leg loops can sometimes feel uncomfortable or restrictive, this wide seat gives clients more space and stability.

## 5. Contents

- Seat with ropes attached. Working load: 400 lbs.
- Instruction Sheet Keep a clear distance of 6 ft. from walls and other obstacale.
- Maintenance Checklist

## 6. How to use

Minimum height and space needed: 2,44 m (8 ft)

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact us.

### Hanging instructions

- To hang the Southpaw Sling Seat, use a Southpaw suspension system – Safety Snap (**13799652**) to attach it to the suspension point.
- For easy adjustment, the use of the Southpaw suspension system – Height Adjuster (**13626652**) is recommended.
- When rotational movement is involved, the Southpaw suspension system – Rotational Device (**13738652**) is required.

## 7. Maintenance

### Monthly checks (record on checklist):

All the following wear points should be checked on a monthly basis and recorded on the checklist:

- Check for fraying of rope loops on Safety Snaps. Check Safety Snaps for wear. ( Snaps not included ).
- Check for loose or missing splice locks \* on ropes. Discontinue use if ropes are frayed, or splice locks \* are loose or missing.
- Check for loose or missing stitches. Discontinue use if loose or missing stitches are found. \* Splice locks are the rope staples near the end of the rope.

### Storage

- Store away from direct sunlight
- Keep away from chemicals and cleaning agents
- Not accessible to clients without supervision
- Do not store on the edge (may damage the rim)





### *Cleaning*

- Consult the infection prevention department before applying cleaning methods
- Check clients for allergies
- Clean vinyl with a damp cloth and soap/disinfectant, then dry
- Vacuum carpet regularly and clean with foam carpet cleaner
- Carpet is not machine washable and must not be submerged in water

## **8. Safety**

The safety of your clients depends on carefully selecting activities that match — and challenge — their physical abilities.

Always use proper spotting and safety techniques. In addition, the layout of the room, correct installation, and regular maintenance of your equipment are critical.

Southpaw equipment is not intended for recreational or outdoor use. Under no circumstances should this equipment be used without supervision.

This document is for example purposes only and in no way replaces local, regional, national, or international safety standards. Please note: it is your responsibility to obtain all relevant safety information that applies to your facility.

For more information, you can visit our website [www.nenko.com](http://www.nenko.com)

