



## 1. Product name

Southpaw cuddle swing senior

## 2. Product code

29353065



## 3. Colour

As shown.

## 4. Short description

The Southpaw Cuddle Swing is a versatile and effective swing that provides deep pressure and vestibular stimulation. Made from soft, stretchable nylon mesh combined with durable Cordura® material, the swing offers a comfortable and supportive space where users can fully relax.

## 5. Contents

1 x Cuddle Swing  
1 x Southpaw Suspension System – Safety Snap  
1 x Instruction Manual  
1 x Maintenance Checklist

## 6. How to use

First, attach the included **Southpaw Safety Snap (13799652)** to the loop at the top of the swing. Then secure this snap to the ceiling suspension point.

If needed, you can use a **Southpaw Height Adjuster (13626652, not included)** to adjust the swing to the correct height for the client. For rotational activities, a **Southpaw Rotational Device (13738652, not included)** must be used.

The Cuddle Swing provides full and even support for the client seated in the swing. Please note that the client may require assistance when entering and exiting the swing.

**SPACE REQUIREMENTS:** Ensure at least 1.8 meters (6 feet) of clear space from walls and other obstacles.

### Maintenance Checklist:

All of the following wear points must be checked monthly and recorded on the enclosed checklist.

- Check the loop for wear at the point where it rubs against the Safety Snap. If wear is found: discontinue use until repaired.
- Check seams for loose or missing stitches. If loose or missing stitches are found: discontinue use until repaired.

### Matting Instructions:

- Always use appropriate crash mats and wall mats with suspended equipment.

### Storage:

- Store out of reach of clients without supervision.
- Protect from direct sunlight.



#### **Cleaning Instructions:**

- Consult your infection control department before applying these procedures.
- Check for client allergies beforehand.
- Machine wash in warm water with mild detergent.
- Tumble dry on low heat.

#### **7. Maintenance**

- Consult your infection control department if necessary.
- Always check for client allergies before cleaning.
- Machine wash in warm water with mild detergent.
- Tumble dry on low heat.
- Warning: Incorrect installation of a forged steel eyebolt (½" or 5/8") may result in serious injury or death. If in doubt, contact us.

#### **8. Safety**

The safety of your clients depends on carefully selecting activities that match — and challenge — their physical abilities.

Always use proper spotting and safety techniques. In addition, the layout of the room, correct installation, and regular maintenance of your equipment are critical.

Southpaw equipment is not intended for recreational or outdoor use. Under no circumstances should this equipment be used without supervision.

This document is for example purposes only and in no way replaces local, regional, national, or international safety standards. Please note: it is your responsibility to obtain all relevant safety information that applies to your facility.

For more information, you can visit our website [www.nenko.com](http://www.nenko.com)

