



1. Product name

Nenko Sensory Equipment Wagon

2. Product code

44503010

3. Colour

white



4. Short description

1. Open the doors of the Snoezelen materials trolley.
2. Hang the fibre optic spray onto the trolley.
3. Take the projector out of the cabinet and place it on the trolley.
4. Take the 3-metre power cable out of the cabinet. Insert one end of the cable into the socket on the outside of the Snoezelen trolley (right-hand side when viewed from the front) and plug the other end into a wall socket in the room where the trolley will be used.
5. Fill the bubble unit to 80% with water, leaving enough space for the bubbles.
6. Plug the other end of the small power cable, which has just been placed on the outside of the trolley, into the bubble unit so that it is also supplied with power.
7. Then open the lower front cabinet of the Snoezelen trolley. Inside this cabinet is a control box with 6 red buttons. These buttons are the on/off switches for the electronic products and components used in and on the Snoezelen trolley. The supplied (and below) diagram clearly shows which button controls which device (from left to right).
Important: Always plug the trolley into the wall socket first and only then switch on the red buttons.
Button 1: Aroma diffuser
Button 2: Fibre optics
Button 3: Bubble unit
Button 4: Space projector
Button 5: Music system
Button 6: Bubble unit
8. Do not forget the supplied cardboard box. This can be found in the lower cabinet. The box contains the remote control for the CD player, a CD, 1 effect wheel and 2 fragrance blends (these fragrances can be used with the aroma diffuser).
9. Place an effect wheel in the space projector and check that all switches on the projector are set to "on".
10. Drip 4 drops of one of the fragrance blends onto the aroma pad located at the bottom of the aroma diffuser. Unscrew the bottom of the diffuser, drip 4 drops of the fragrance of your choice onto the aroma pad, and screw the bottom back on.
11. The Snoezelen trolley is now ready for use.

5. Sensory stimulation

Touch, smell, hearing and sight

6. Specifications

Dimensions: 94 x 62 x 127 cm

Power supply: 220 V





7. Included

- 1 x Snoezelen materials trolley
- 1 x Aroma diffuser
- 1 x HiFi music system with CD player
- 1 x Light source
- 1 x Fibre optic spray
- 1 x Bubble unit
- 1 x Projector
- 4 x Storage trays
- 1 x Sensory set – Hearing
- 1 x Sensory set – Seeing
- 1 x Sensory set – Feeling
- 1 x Sensory set – Smelling

For more information, you can visit our website www.nenko.com

