



## 1. Name of the product

Weighted lap cushion with balls (5 KG)

## 2. Product code

59287872

## 3. Colour

See image



## 4. Brief description

Maya is a 3-in-1 weighted lap cushion that provides calmness in three functional ways. On the one hand, this weighted cushion helps to prevent the user from becoming restless. But on the other hand, this cushion gives extra rest because restless hands can be placed in the pouch.

In the pockets are two balls, which you can move with your fingers. Restless users are provided in their fidgeting needs. Considering the balls can be moved for stimulation, a signal will automatically be sent to the brain by moving the balls. Moving the balls contributes to better concentration and relaxation.

## 5. How to use

1. Place the pouch cushion on the patient's lap to relax.
2. Place the user's hands in the pouch and discover the surprises in the pockets!

## 6. Sensory Stimulations

Touch, proprioception

## 7. Specifications

Size: 66 x 17 x 6 cm

Weight: 5 KG

## 8. Included

1 x Weighted pouch lap cushion with balls

## 9. Maintenance

1. The lap cushion can be cleaned with a soft, damp cloth and a bit of soap.
2. After cleaning with the soap, we recommend wiping the blanket with a damp, clean cloth.
3. This prevents damage to the fabric.

For more information, you can visit our website [www.nenko.com](http://www.nenko.com)

